# A 814

- de** Gebrauchsanweisung Aufnahme für Air Scaler und Piezo Scaler Spitzen
- da** Brugsanvisning Holder til Air Scaler og Piezo Scaler spidser
- en** Operating instructions Holder for the tips of air scalers and piezo scalers
- es** Instrucciones de manejo Alojamiento para puntas para Air Scaler y Piezo Scaler
- fi** Käyttöohje Pidike ilmatoimisten ja pietsosähköisten hammaskivenpoistolaitteiden kärkien käsittelyyn.
- fr** Mode d'emploi Logement pour pointes pour Air Scaler et Piezo Scaler
- hr** Upute za uporabu Prihvat za Air Scaler i Piezo Scaler vrhove
- it** Istruzioni d'uso Supporto per punte Air Scaler e Piezo Scaler
- nl** Gebruiksaanwijzing houder voor punten van Air Scalers en Piezo Scalers
- no** Bruksanvisning holder for Air scaler og Piezo scaler-spisser
- pt** Instruções de utilização Suporte para ponteiras para Air Scaler e Piezo Scaler
- sl** Navodila za uporabo Nastavek za konice Air Scaler in Piezo Scaler
- sv** Bruksanvisning Hållare för Air Scaler och Piezo Scaler spetsar

**de** ..... 4  
**da** ..... 14  
**en** ..... 24  
**es** ..... 34  
**fi** ..... 44  
**fr** ..... 54  
**hr** ..... 64  
**it** ..... 74  
**nl** ..... 84  
**no** ..... 94  
**pt** .....104  
**sl** .....114  
**sv** .....124

---

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---

<b>Notes about these instructions</b> .....	24
<b>Intended use</b> .....	25
Queries and technical problems.....	25
<b>Items supplied</b> .....	26
Optional accessories.....	26
Disposal of the packing material .....	26
<b>Warning and Safety instructions</b> .....	27
<b>Areas of application</b> .....	28
Installation .....	28
A 812 and A 813 adapters.....	28
Installation on mobile units and baskets with silicone adapters .....	28
Filters.....	29
A 813 adapter.....	29
A 812 adapter.....	29
Reprocessing .....	30
After reprocessing .....	31
Cleaning or replacing filters.....	32

### Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

### Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

### Additional information and comments

Additional information and comments are contained in a simple frame.

### Operating steps

Operating steps are indicated by a black square bullet point.

#### Example:

- Select an option using the arrow buttons and save your choice with *OK*.

### Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

#### Example:

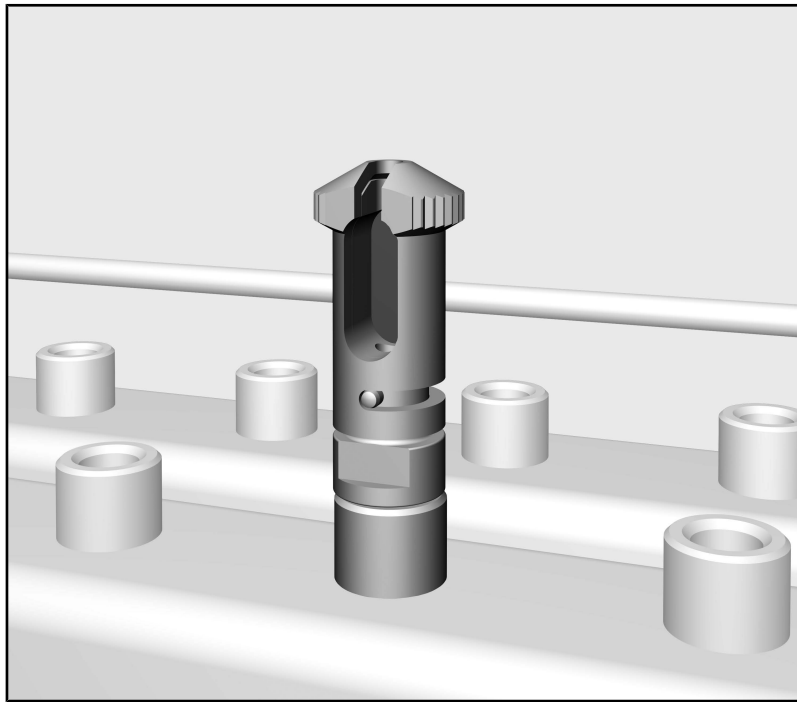
Settings  menu.

The A 814 holder enables the tips of air scalers and piezo scalers to be reprocessed in a Miele washer-disinfector. Follow the operating instructions for your washer-disinfector as well as the instructions of the medical product manufacturer on how to reprocess their items by machine.

### **Queries and technical problems**

If you have any queries or technical problems that you would like to discuss please contact Miele. Contact details can be found at the back of the Operating Instructions for your washer-disinfector or go to [www.miele-professional.com](http://www.miele-professional.com).

- A 814 holder with screw cap, height approx. 41 mm, Ø 16 mm

### **Optional accessories**

The following and other accessories are available to order from Miele:

- A 812, two-piece adapter with 4 screw threads for various rinse fittings, height approx. 48 mm, Ø 45 mm, 4 blind stoppers.
- A 813, two-piece adapter with 4 screw threads for various rinse fittings, height approx. 48 mm, Ø 45 mm, 4 nozzles with silicone hoses and 3 blind stoppers.
- E 197, mesh tray with lockable lid and 2 holding lugs for instruments, height 42 mm, width 150 mm, depth 225 mm
- E 362, blind stopper
- FP 20, 20 filter plates, Ø 30 mm

### **Disposal of the packing material**

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

## en - Warning and Safety instructions

---

Read the operating instructions carefully before using this holder. This prevents both personal injury and damage to the rinse fitting. Keep these operating instructions in a safe place.

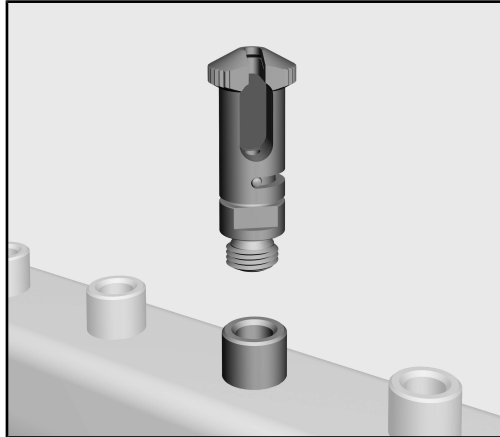
 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ The holder is approved solely for the applications specified in the "Application technology" chapter of these operating instructions. All other uses, conversions, and modifications are not permitted.
- ▶ Inspect all mobile units, baskets, modules and inserts daily as described in the "Maintenance" section in the Operating instructions for the washer-disinfector.
- ▶ New load carriers must be cleaned in the washer-disinfector without a load prior to first use.
- ▶ To clean and disinfect hollow instruments thoroughly from the inside they must be attached to the relevant holders or injector nozzles on the injector manifold. The connectors must not come loose during a programme!  
Hollow instruments which have come away from their connectors during the programme must be processed again.
- ▶ When handling tips for air scalers and piezo scalers, be careful not to injure yourself on the functional ends of the instruments. For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments.
- ▶ Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be re-processed.
- ▶ The last rinse should be carried out with demineralised water wherever possible.

Miele cannot be held liable for damage caused by non-compliance with these Warning and Safety Instructions.

### Installation

- Remove a nozzle from the injector manifold of your upper basket or mobile unit.


- Screw the holder in at this position. Tighten the holder with the aid of an open spanner (width across flats 9).

### A 812 and A 813 adapters

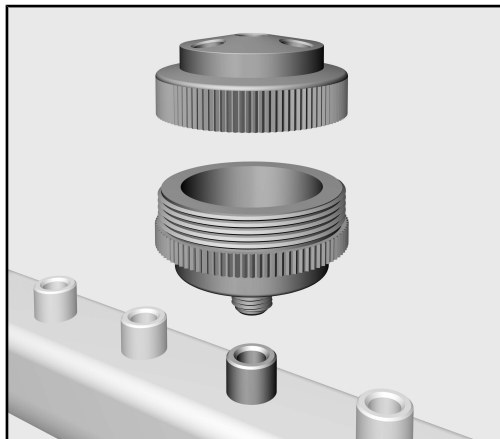To increase capacity, you can combine the A 814 holder with the A 812 or A 813 adapters. Both adapters each have 4 ports. The A 814 holder may be fitted to a maximum of 3 ports. The 4th port must always be sealed with a blind stopper.

- The A 812 adapter can be used on injector manifolds with silicone adapters.
- The A 813 adapter is intended for use on injector manifolds with screw connections.

- Follow the fitting instructions in the operating instructions for the A 812 and A 813 adapters.
- Screw the A 814 holders into the ports on the adapters.

⚠ Ports that are not in use must be sealed with a blind stopper. The 4th port must always be sealed with a blind stopper.


### Installation on mobile units and baskets with silicone adapters

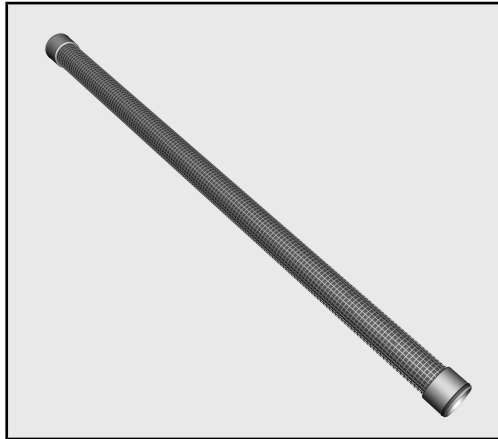The A 814 holder must be firmly screwed in place. In the case of baskets and mobile units with silicone adapters, an A 812 adapter must be used that can then be fitted with up to 3 A 814 holders.

⚠ Inserting the A 814 holder into a silicone adapter is not permitted.

### Filters

When the A 814 holder is used, the wash water must be filtered.

- The injector manifold must be fitted with an A 800 tubular filter.

The tubular filter is supplied with its own operating instructions. Follow the instructions on how to perform regular cleaning on the tubular filter.

### A 813 adapter

When the A 813 adapter is used, an FP 20 filter plate can be inserted into the adapter instead of the tubular filter.

Do not insert a filter plate into the adapter if the injector manifold has been fitted with an A 800 tubular filter.

### A 812 adapter

Injector manifolds with silicone adapters cannot be fitted with a tubular filter. The A 812 adapter must therefore always be fitted with an FP 20 filter plate for this application. Observe the operating instructions for the adapter.

### Reprocessing

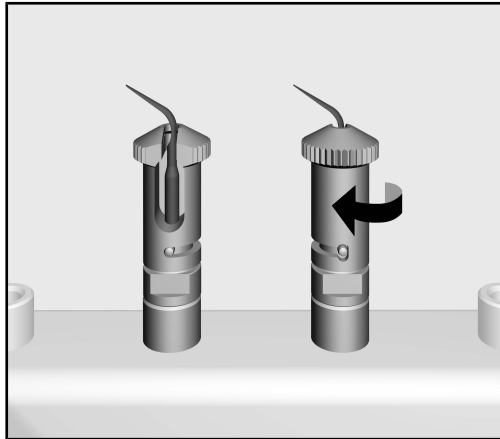
For validation purposes please make sure that you follow the loading instructions given on the template.

⚠ Any hollow items that have become disconnected from their injector nozzles, adapters or holders during processing must be reprocessed.

⚠ Danger of injury: When handling tips for air scalers and piezo scalers, be careful not to injure yourself on the functional ends of the instruments. For your own safety, wear gloves and, if necessary, a surgical mask and protective goggles when sorting the instruments.

To minimise the risk of injury, the inserts should be loaded from the back forwards and unloaded in the reverse order. Wear clean gloves when removing the inserts.

- Remove the tip from the transmission instrument. Use the tools recommended by the instrument manufacturer, such as a torque spanner.
- Remove the tip from the instrument.
- If the instrument can be reprocessed by machine, place it in a lockable mesh tray, such as the E 197. Instruments with hollow spaces must be sorted with the opening facing downwards so that the water can enter and escape unhindered. Place the mesh tray in a free space in the upper or lower basket or in the mobile unit or module.
- Coarse soiling on the tips must be rinsed off under running water. Use the brushes recommended by the manufacturer.
- Check that the cooling channel is free of blockages. The instrument manufacturers provide appropriate tools, such as brushes and cleaning wires, for this work. Blocked instruments must not continue to be used.
- Instruments pretreated with cleaning or disinfection agents must be rinsed, for example under running water, before they are reprocessed in the washer-disinfector. Alternatively, you can start the Rinsing programme before starting the reprocessing programme.
- At the grip end of the A 814 holder, turn the screw cap anti-clockwise as far as it will go.

### After repro- cessing

- Pick up the tip at the sharp end and place it sideways into the A 814 holder.
- Turn the screw cap of the holder clockwise as far as it will go. The tip is then encapsulated in the holder.
- At the grip end of the A 814 holder, turn the screw cap anti-clockwise as far as it will go.
- Take hold of the tip at the sharp end and remove it sideways from the holder.
- Perform a visual inspection to check the cleanliness of the tip.
- After reprocessing, dry the tip – and the cooling channel in particular – with sterilised compressed air, then sterilise it following the manufacturer's instructions. Observe national health and safety regulations.

## en - Areas of application

### Cleaning or replacing filters

#### Cleaning the A 800 tubular filter

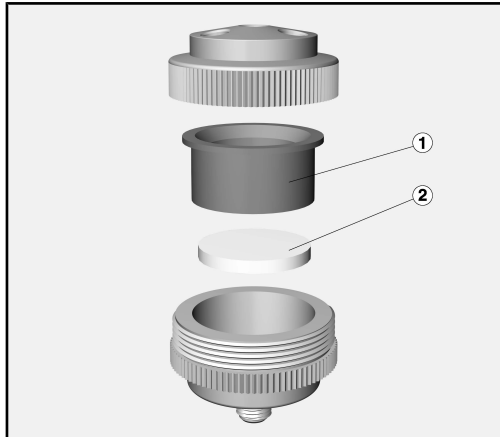
The tubular filter must be cleaned **at least once a week**.

- Follow the cleaning instructions in the operating instructions for the tubular filter.

#### Replacing FP 20 filter plates

Borosilicate filter plates must be replaced **approx. every 2 weeks** or **after 20 cycles** have been run.

- To do this, unscrew the adapter.
- Unscrew the top part from the bottom part.

- Remove the silicone adapter ① and the filter plate ② from the bottom part.
- Rinse out the bottom part.
- Insert a new FP 20 filter plate into the bottom part. The plate must rest fully on the ring-shaped support of the bottom part.

⚠ Damaged or used filter plates must not be inserted under any circumstances.

- Insert the silicone adapter ① into the bottom part.
- Screw the top part and bottom part together.
- Screw the adapter back onto the injector manifold.

Do not insert a filter plate into the holder if the injector manifold has been fitted with an A 800 tubular filter.

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